


HCG – Pregnancy Rapid Test Strip

INTERPRETATION OF RESULTS

POSITIVE

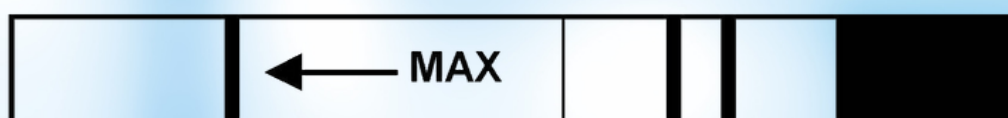

NEGATIVE



INVALID



DIRECTIONS FOR USE

Allow test strip and urine samples to equilibrate to room temperature (20-30 oC) prior to testing.

1. Remove the hCG-Pregnancy Rapid Test Strip from Pouch. Use strip as soon as possible but within 1 hour after removal from pouch especially if the room temperature is more than 30°C and in high humidity environment.

2. Immerse the test strip in the urine sample with printed sample end pointing toward the urine for at least 5 seconds. Be sure the sample level is below the marked sample line on test strip.

3. Wait for red bands to appear. The test should be read in approximately 3-5 minutes for urine.

Negative: Only one pink colored line appears in the Control area.

Positive: In addition to the colored line in the control region a clearly distinguishable pink.

In conclusive: If no line appears in the control as well as the test region, the test should be repeated with fresh test strip.

HCG – Pregnancy Rapid Test Strip

HCG – Pregnancy Rapid Test Strip



Sensitivity and Specificity

SENSITIVITY - The analytical sensitivity of the hCG-Pregnancy Rapid Test Strip is 25mIU/ml.

SPECIFICITY - Negative results were obtained from all tests conducted with 500 mIU/ml hLH, 1000 mIU/ml hFSH and 1000 IU/ml hTSH.

Corporate Office:-

C/609, 6th Floor, Mittal Commercial, Unit 2,
Hasan Pada Road, Chimatpada Marol, Andheri
(E), Mumbai - 400059.

Factory Address:-

G/F- 12/13, Evershine-2, Survey NO.
307/3/1, Balitha, N.H No 48, Vapi,
Valsad, Gujarat, 396191.