



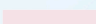
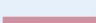
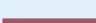
NT-ProBNP Rapid Test Kit (Serum/Plasma/Whole Blood)

INTERPRETATION OF RESULTS

NT-ProBNP Color Chart

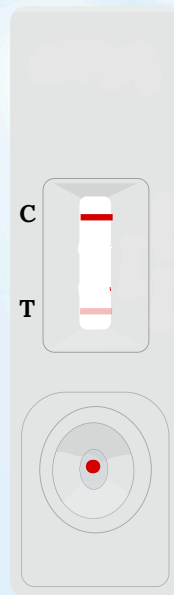

Positive

Test Line	Interpretation
NIL	Negative
	>450pg/ml
	>1500pg/ml
	>2500pg/ml
	>5000pg/ml

DIRECTIONS FOR USE

Allow the test, specimen, buffer and/or controls to reach room temperature (15-30°C) prior to testing.

1. Bring the pouch to room temperature before opening it. Remove the test cassette from the sealed pouch and use it within one hour.

3. Wait for the colored line(s) to appear.

Read results at 10 minutes.

Do not interpret the result after 20 minutes.

2. Place the cassette on a clean and level surface.

For Serum or Plasma specimen:

· Hold the dropper vertically and transfer 3 drops of serum or plasma (approximately 75µL) to the sample well of the test cassette, then start the timer immediately. See illustration below.

For Venipuncture Whole Blood specimen:


· Hold the dropper vertically and transfer 3 drops of whole blood (approximately 75µL) to the specimen well of the test cassette, then add 1 drop of buffer (approximately 40µL), and start the timer. See illustration below.

For Fingerstick Whole Blood specimen:


· To use a capillary tube: Fill the capillary tube and transfer approximately 75µL of fingerstick whole blood specimen to the specimen well of test cassette, then add 1 drop of buffer (approximately 40µL) and start the timer. See illustration below.

· To use hanging drops: Allow 3 hanging drops of fingerstick whole blood specimen (approximately 75µL) to fall into the specimen well of test cassette, then add 1 drop of buffer (approximately 40µL) and start the timer. See illustration below.

NT-ProBNP Rapid Test Kit (Serum/Plasma/Whole Blood)

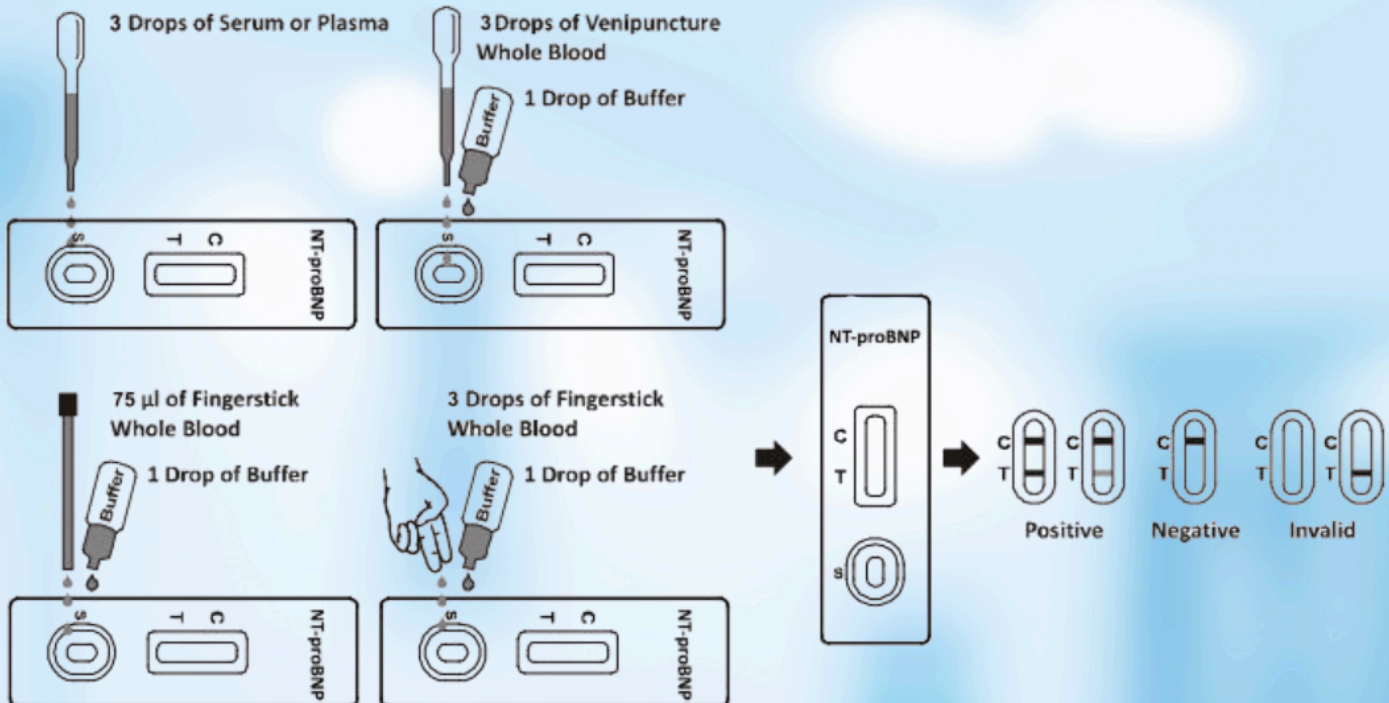

Cat. No.	Product Description	Specimen	Format	Kit Size	Cut-Off	Status
CBN-402	NT-proBNP Rapid Test Cassette	WB/S/P	Cassette	10 T	0.45 ng/mL	CE

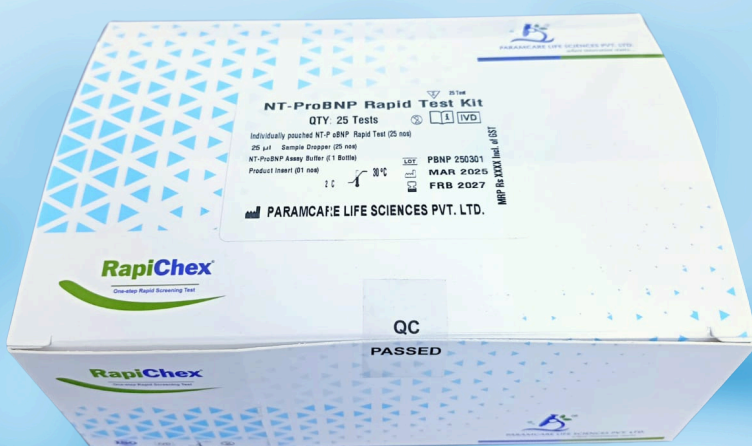

NT-ProBNP Rapid Test Kit

Corporate Office:-

C/609, 6th Floor, Mittal Commercial, Unit 2,
Hasan Pada Road, Chimatpada Marol, Andheri (E),
Mumbai - 400059.

Factory Address:-

G/F- 12/13, Evershine-2, Survey NO.
307/3/1, Balitha, N.H No 48, Vapi, Valsad,
Gujarat, 396191.