


TSH Rapid Test Kit (Serum/Plasma/Whole Blood)


INTERPRETATION OF RESULTS

Figure 1

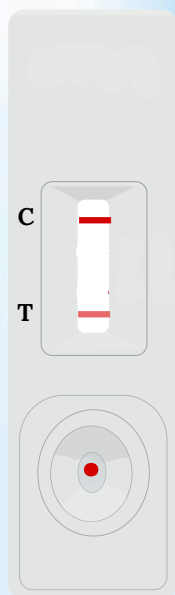


Positive

Figure 2

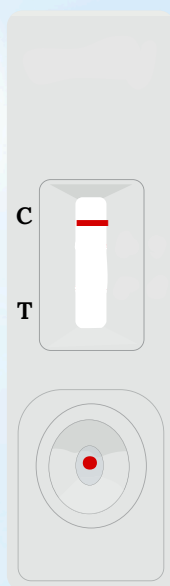

Negative

Figure 3

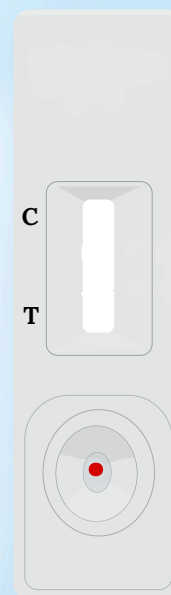

Invalid

DIRECTIONS FOR USE

Allow the test device, specimen to equilibrate at room temperature (15-30°C) before testing.

1. Bring the pouch to room temperature before opening it. Remove the test cassette from the sealed pouch and use it within 1 hour.
2. Place the cassette on a clean and level surface.
3. For Serum or Plasma specimen: Hold the dropper vertically and transfer 1 drop (25 µL) drops of serum or plasma and add 1 drop of buffer (approximately 40 µL) into the specimen well, and start the timer. See illustration below. For Venipuncture Whole Blood specimen: Hold the dropper vertically and transfer 1 drop (25 µL) drops of whole blood and add 1 drop of buffer (approximately 40 µL) into the specimen well, and start the timer. For Fingerstick Whole Blood specimen: Take sample using sample dropper and transfer approximately 25 µL (1 drops) of fingerstick whole blood specimen to the specimen well of test cassette, then add 1 drop of buffer (approximately 40 µL) and start the timer
4. Note: Do not interpret the result after 20 minutes.

INTERPRETATION OF RESULTS

Positive: The control line (C) and test line (T) lines are visible on the test device. This is positive for TSH hormone.

Negative: The control line is the only visible line on the test device. No TSH hormone were detected.

Invalid: The control line fails to appear.

RapiChexTM



PARAMCARE LIFE SCIENCES PVT. LTD.
where innovation starts...

TSH Rapid Test Kit (Serum/Plasma/Whole Blood)



Sensitivity and Specificity

Method		Other Rapid Test		Total Test
TSH Rapid Card Test	Result	Positive	Negative	
	Positive	75	0	
	Negative	0	230	
Total Results		75	230	305

SENSITIVITY : >98% (95% CI*: 94.4%-99.8%)
SPECIFICITY : >98% (95%CI*: 96.7%-99.6%)

Corporate Office:-

C/609, 6th Floor, Mittal Commercial, Unit 2,
Hasan Pada Road, Chimatpada Marol,
Andheri (E), Mumbai - 400059.

Factory Address:-

G/F- 12/13, Evershine-2, Survey NO.
307/3/1, Balitha, N.H No 48, Vapi, Valsad,
Gujarat, 396191.