


## HCG Pregnancy Rapid Test Kit (Serum/Plasma/Whole Blood)


### INTERPRETATION OF RESULTS

Figure 1

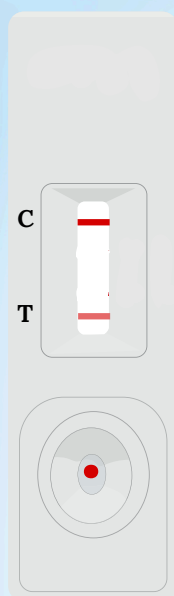


Positive

Figure 2

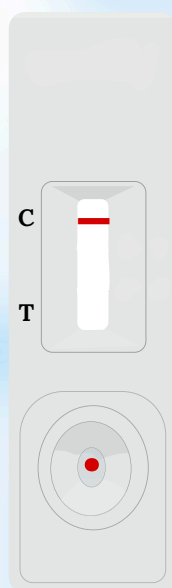

Negative

Figure 3

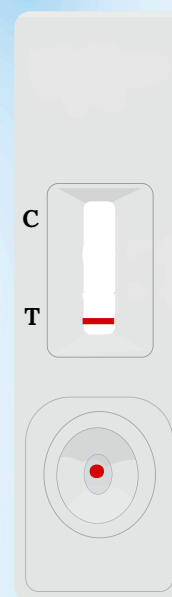

Invalid

### DIRECTIONS FOR USE

Allow the test device, specimen and/or buffer to equilibrate at room temperature (15-30°C) before testing.

1. Remove the test from the pouch by tearing from the notch.
2. Place the device immediately on a clean horizontal surface.
3. Add 1 drop 25µL specimen (serum/plasma/whole blood) into the Sample window 'S' with the given dropper.
4. Add 1 drop 40 µL of assay Buffer and start the timer.
5. Read the results at 5 minutes. Do not read after 10 minutes.

### INTERPRETATION OF RESULTS

1) Positive: Both test line (T) and control line (C) appear in the result window. The intensity of the color in the test line regions may vary depending on the concentration of the HCG in the sample. A faint line (T) should be considered as positive result. Neither the quantitative value nor the rate of increase in HCG can be determined by this qualitative test.

2) Negative: Only the control line (C) appears in the result window. The absence of a test line (T) indicates a negative result.

3) Invalid: If the control line (C) does not appear, the test result is invalid. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test.

# RapiChex<sup>®</sup>



PARAMCARE LIFE SCIENCES PVT. LTD.  
where innovation starts....

## HCG Pregnancy Rapid Test Kit (Serum/Plasma/Whole Blood)

### HCG Pregnancy Rapid Test Kit



### Sensitivity and Specificity

Method		Other hCG Rapid Test		Total Results
hCG Pregnancy Rapid Test	Results	Positive	Negative	
	Positive	163	0	163
	Negative	0	280	280
Total Results		163	280	443

**Relative Sensitivity: 99.9%**

**Relative Specificity: 99.9%, Overall Accuracy: 99.9%**

### Corporate Office:-

C/609, 6th Floor, Mittal Commercial, Unit 2,  
Hasan Pada Road, Chimatpada Marol, Andheri (E),  
Mumbai - 400059.

### Factory Address:-

G/F- 12/13, Evershine-2, Survey NO.  
307/3/1, Balitha, N.H No 48, Vapi, Valsad,  
Gujarat, 396191.