


HCG – Pregnancy Rapid Test Strip (DIP STICK)

INTERPRETATION OF RESULTS

IMMERSE DIPSTICK

READ RESULTS AT 3 MINUTES FOR URINE OR 5 MINUTES FOR SERUM

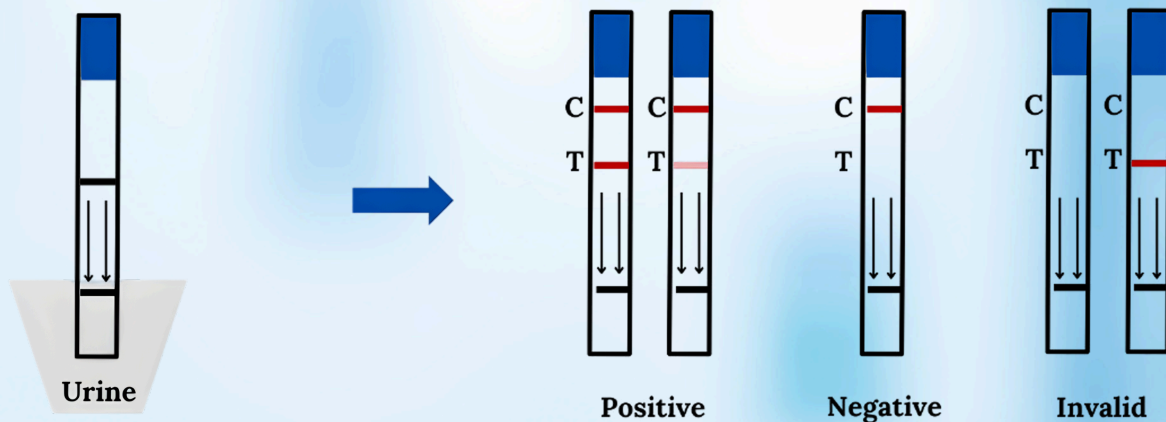

Human Chorionic Gonadotropin Rapid Test (Urine)

DIRECTIONS FOR USE

Read the entire procedure carefully prior to performing any tests.

1. Allow Test strip and urine samples to equilibrate to room temperature (20-30°C) prior to testing.
2. Remove the Test strip from the foil pouch and test it immediately in one hour.
3. Hold the Test strip so as to place the absorbent tip ($\geq 2/3$) in urine in a clean cup for at least 20 seconds.
4. Then lay down the product on a clean and stable desk, start the timer immediately.

INTERPRETATION OF RESULTS

Negative: Only one pink colored line appears in the Control area showing negative for HCG.

Positive: In addition to the coloured line in the control region a clearly distinguishable pink - rose coloured line also appears in the test region indicating a positive result and that the sample contains HCG.

In conclusive / Invalid: If no line appears in the control as well as the test region, the test should be repeated with fresh card. If no line appears in the control region and line appears in the test region then also its invalid and test should be repeated.

HCG – Pregnancy Rapid Test Strip (DIP STICK)

HCG – Pregnancy Rapid Test Strip



Sensitivity and Specificity

Method		Other hCG Rapid Test		Total Results
hCG Pregnancy Rapid Test	Results	Positive	Negative	
hCG Pregnancy Rapid Test	Positive	163	0	163
	Negative	0	280	280
Total Results		163	280	543

Relative Sensitivity: >99.9% (98.7%~100%)

Relative Specificity: >99.9% (99.2%~100%)

Overall Accuracy: >99.9% (99.5%~100%)

95% Confidence Intervals

Corporate Office:-

C/609, 6th Floor, Mittal Commercial, Unit 2,
Hasan Pada Road, Chimatpada Marol, Andheri
(E), Mumbai - 400059.

Factory Address:-

G/F- 12/13, Evershine-2, Survey NO.
307/3/1, Balitha, N.H No 48, Vapi,
Valsad, Gujarat, 396191.